

LUBRI-GEL LUBRICATING GEL

DESCRIPTION

Clear, greaseless, water soluble gel preserved with methyl hydroxybenzoate 0.2% w/w. No deleterious effect on rubber, metal or plastics.

ACTION

Lubricant

INDICATIONS

Lubricating gel for personal, gynaecological and surgical use. For use on catheters, specula, probes, electrodes and other instruments. For use on gloves or hands in medical examinations. For vaginal lubrication prior to sexual intercourse and use with condoms.

CONTRAINDICATIONS

Known hypersensitivity to methyl hydroxybenzoate.

DIRECTIONS

Remove cap and peel off seal. Squeeze tube to obtain desired amount. Reapply as required. Replace cap after use.

ADVERSE REACTIONS

Well tolerated. Should irritation occur, discontinue.

PRESENTATION

Tubes: 50g, 100g

STORAGE

Store below 30°C.

POISONS SCHEDULE

Unscheduled.